

Oosafe®

# FILTERS

**CE** Manufactured in accordance  
with ISO 9001:2015  
and EN ISO 13485:2016






Oosafe® Filter 4 and Oosafe® Filter 6 are medical device inline filters used in between CO<sub>2</sub> incubators and a gas source to purify incoming CO<sub>2</sub>, N<sub>2</sub> and premixed gas against volatile organic compounds (VOCs), chemically active compounds (CACs), microorganisms, dust and other harmful compounds and particles.

- **Better Embryo Development:** Purified CO<sub>2</sub>, N<sub>2</sub> and premixed gas results in better embryo development.
- **Removal of VOCs, microorganisms and particles:** The combination of activated carbon granules, HEPA filter and pre-filters ensures high VOC removal capacity and traps microorganisms and particles (H14 efficiency ≥ 99.995%).
- **Strict Quality Control:** Each filter is produced in a cleanroom in fully controlled environment and pressure tested to ensure product safety.
- **Replacement:** Replacement is recommended every 4 months for Oosafe® Filter 4 and every 6 months for Oosafe® Filter 6.
- **Approved by Scientists:** Oosafe® Filters have been used for many years in well-known IVF clinics all over the world as the main gas purification tool.
- **Traceability:** Each filter's manufacturing history can be traced easily based on the lot number written on the label.
- **High Product Quality:** Manufactured in accordance with ISO 9001:2015 and EN ISO 13485:2016 and classified as medical device class I according to MDR (EU) 2017/745.
- **Packaging:** Each filter is individually packed and sealed. Product is sold in a cardboard box as four pieces set, including certificate of conformity and installation instruction.




Oosafe® FILTER 4  
(Replace every 4 months)

Product reference:  
**OOIF-SM04**

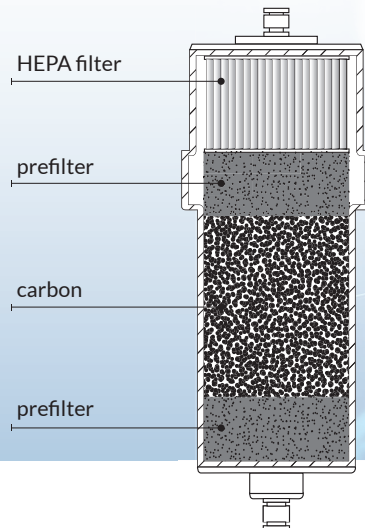

Oosafe® FILTER 6  
(Replace every 6 months)

Product reference:  
**OOIF-ST06**

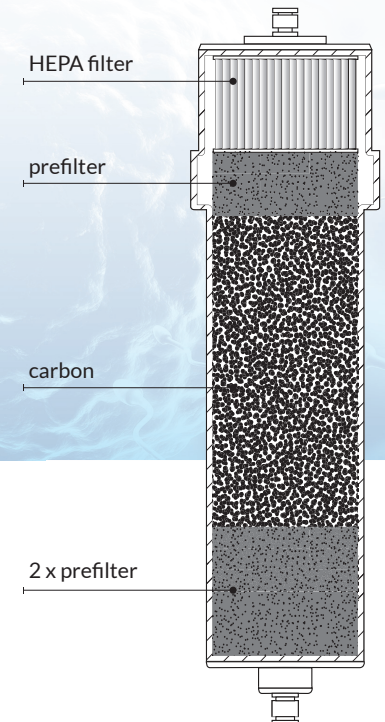


Oosafe®

FILTERS

## OOSAFE® FILTER CONNECTION SET FOR BENCHTOP INCUBATORS

Leak-free connection set for installation of Oosafe® Filter 4 and Oosafe® Filter 6 to the benchtop incubators using metal gas pipes, ie. Cook Minc/Origio Planer benchtop Incubators.

- **Easy to Install:** Connection set is easily connectable to the filter, incubator and gas source.
- **Ready to Use:** Comes with all necessary connectors and tubing.
- **Original Connectors:** Supplied with the same type of connectors as metal gas pipe manufacturer to ensure no gas leak or connection problems.
- **Connection Set includes:** 2 x Polyethylene 6 mm hard tubing (50 cm long each), 2 x 6 mm connectors, 2 x Swagelok metal connectors.

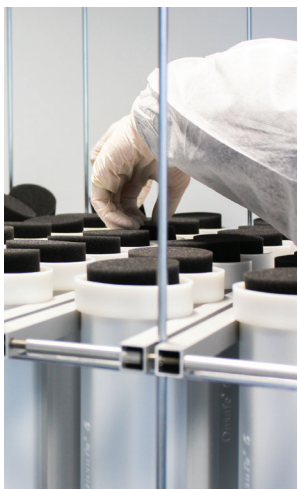

OOSAFE® FILTER CONNECTION SET FOR BENCHTOP INCUBATORS  
Product reference: **OOFCS-MC**



Each filter is pressure tested.





## PRODUCTS TO BE USED WITH OOSAFE® FILTERS

For the best performance, use only original products!

- Magnet Holder is designed to securely hold Oosafe® Filters. Prevents the possibility of any flow restriction occurring to improper filter placement. The holder is magnetic and can be placed on the side of the incubator or any metallic surface.
- Connectors for Oosafe® Filters are designed with shut/off valve that filters can be easily changed even with the gas flow on.
- Connectors are available in two sizes: 6 mm hard tubing and 8 mm soft tubing.



MAGNET HOLDER  
for Oosafe® Filters  
Product reference: **OOHLD-01**



CONNECTOR for Oosafe® Filters  
6 mm diameter hard tubing  
Product reference: **AS-852086**



CONNECTOR for Oosafe® Filters  
8 mm diameter soft tubing  
Product reference: **AS-852085**

## ORDER INFORMATION

Product reference	Product name
OOIF-SM04	Oosafe® Filter 4
OOIF-ST06	Oosafe® Filter 6
OOFCS-MC	Oosafe® Filter Connection Set for Benchtop Incubators
OOHLD-01	Magnet Holder for Oosafe® Filters
AS-852086	Connector for Oosafe® Filters, 6 mm diameter hard tubing
AS-852085	Connector for Oosafe® Filters, 8 mm diameter soft tubing

